

## How to create a Saved phrases page Grid 3

This video will show you how to create a saved phrases page. The first thing I need to do, is make my cells a bit bigger, because we are going to be putting whole phrases in them. In order to do that I'm going to click on the cell and then drag the corner along so the cell expands. I'm going to do that with the rest of the cells in the row. You can keep doing that for the whole page or you can just copy the first row and then paste it on the rest of the page just to save time.

So, after I have expanded my cells, I want to create a wordlist. To do that I'm going to select all the cells apart from the cells in the last row. Then I'm going to click on "create cell". From here I want to select the "wordlist command". If your wordlist looks like that you can click on edit and then just click on "remove" until your wordlist is empty, and then press "ok".

Now that you have your wordlist, click on one of the cells in the last row, and then click on "create cell". We want to have a command that allows us to add phrases onto the page. So, we are going to click here and search for "add". From the options below, we are going to click on "add word". We are going to do the same on another cell on the last row. We are going to click on it, and then we are going to "create cell". Now we want a command that allows us to remove phrases from the page, so we are going to search for "remove". Now we are going to click on "remove word". Now we can name them add phrase and remove phrase accordingly. So click on the "add word" cell, and click on change label, then type add phrase. Do the same over here and type remove phrase. What we want to do usually is give the add phrase cell a nice green colour, so we are going to "styles", and then we are going to "fill", and select a nice green colour.

We can do the same on remove phrase, but we are going to give it a nice red colour instead. So, we click on "fill" again and then choose a nice red colour. In order to save your changes, click on "finish editing", and then "yes".

You can see now, that our page is finished. We can test it by jumping back home and then make a phrase and save it. ....so, our phrase is "I want your playstation", so we can now go back to our saved phrases – I have saved mine under news – and we can press, "add phrase". So, you can see that there is a cell highlighted here, click on it, and you can see that your phrase is now saved. If you want to remove a phrase, just click on "remove phrase", you can see that the cell is being highlighted red. To remove it completely, just click on it. And you can see now that the phrase is gone.